Sample Guide - Bloods

Different sample types, blood tubes required and their main indications for use. Always ensure tubes are correctly filled (to the line) and in-date

E	EDTA (Pink top)	Anti-coagulated whole blood	Haematology	
EP	EDTA plasma (Pink top)	Separated/ centrifuged whole blood	*Hormones	
Cit	Sodium Citrate (Green top)	Anti-coagulated whole blood	Coagulation	
нв	Heparin Whole Blood (Orange top)	Anti-coagulated whole blood (HB)	Reptiles/Avians	base On
HP	Heparin Plasma (Orange top)	Separated/ centrifuged whole blood	**Biochemistry	

^{*}Unstable hormones or enzymes, eg ACTH, PTH, Renin

^{**} In general serum is the preferred sample for biochemistry

OxF	Oxalate Flouride (Yellow top)	Anti-coagulated whole blood	Glucose	MARK GO
SS	Separated Serum (White top)	Clotted blood	Biochemistry	100 100 100 100 100 100 100 100 100 100
SG	Spun Gel (Brown top)	Clotted blood	Biochemistry	12015-09 LOT 40775
Р	Plain Tube (White top)	Clotted blood/serum if separated	Biochemistry	100 100 100 100 100 100 100 100 100 100

Notes

- For haematology submit anti-coagulated whole blood (EDTA, Citrate, HB) and 2 fresh, air dried smears
- Serum (SS/SG) is the **preferred** sample for biochemistry for all species
- For small patients, avian and reptiles, heparinised blood (HB) is the most **convenient** sample since it can be used for haematology and biochemistry; 2 fresh (not bottle) air dried smear are also required
- Glucose Testing Blood glucose measurement should be performed on fasted blood whenever possible. A separated serum/spun gel is the preferred sample type for Glucose measurement. Serum/spun gel separation should be performed within 30 minutes of taking the sample. Fluoride Oxalate tubes may also be used for Glucose measurement.

- Samples which require special handling (PTH, PTHrP, ACTH) see page 23
- Serum Gel tubes should not be used for therapeutic monitoring, hormone assays or ionised calcium (iCa) estimation
- If sample volume is inadequate for all of the test in a profile, please prioritise the analytes accordingly otherwise they will be processed in a standard order
- Sample supply tubes for large animals are also available from the laboratory.